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Droga Test Marijuana (THC)

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Drug Test Marijuana (THC)

To perform with urine Result in 5 minutes

INTENDED USE AND SUMMARY

Marijuana (THC) Rapid Test is a side flow chromatographic immunoassay for the detection of 11-nor- Δ^9 -THC-9 COOH (THC metabolite) in human urine at a limit concentration of 50 ng/ml. This test will detect other related compounds, please refer to the table of analytical specificity in this leaflet. This test provides only a preliminary result of the analytical test. It is necessary to use a more specific alternative chemical method to obtain a confirmed analytical result. Gas chromatography and mass spectrometry (GC/MS) is the method of confirmation to be preferred. Any findings for the determination of the presence of drug abuse should be related to clinical considerations and professional judgment, especially when the preliminary result is positive.

THC (tetrahydrocannabinol) is the active ingredient in cannabis (marijuana). When smoked or administered orally, it produces euphoric effects. Users have compromised short-term memory and slow learning.

Users can also experience transient episodes of confusion and anxiety. A relatively heavy long-term use may be associated with behavioral disorders. The peak effect of marijuana smoking occurs in 20-30 minutes and the duration is 90-120 minutes after a cigarette.

Elevated levels of urinary metabolites occur within a few hours of exposure and remain detectable for 3-10 days after intake. The test uses a monoclonal antibody to selectively detect elevated levels of marijuana in the urine. The Disposable Marijuana Rapid Test (THC) gives a positive result when the concentration of Marijuana in the urine exceeds 50 ng/ml. This is the screening limit suggested for positive samples established by the Substance Abuse and Mental Health Service Administration (SAMHSA, USA).

PRINCIPLE

The drug test Marijuana (THC) is a rapid immunochromatographic based on the principle of competitive binding. Drugs that might be present in the urine sample compete with its conjugate for the same antibody binding sites.

During the test, a sample of urine migrates upwards by capillary action. Marijuana, if present in the urine sample below 50 ng/ml, will not saturate the binding sites of antibody coated particles in the test strip.

Antibody-related particles will be captured by the immobilized conjugate and a visible colored band will appear in the test area. The colored band will not appear in its area if the level of the drug present within the urine sample will be higher than its cut-off, as all relevant antibody binding sites will be saturated.

A test drug positive urine sample will not cause the formation of a colored band due to drug competition, while a test drug negative urine sample or a sample containing a drug concentration at the cut-off will cause the formation of the colored band in the specific test area.

As a control of the procedure, a colored band will appear in the relative area, indicating that a correct amount of sample has been used and that the migration on the membrane has taken place.

COMPOSITION

Test for detection of disposable marijuana, instructions for use, jar for collecting samples.

STORAGE AND HANDLING

- Store the test in a cool, dry place at a temperature between 2 and 30° C. Keep away from light. Exposure to temperature and/or humidity outside the specified conditions may cause inaccurate results.
- Do not freeze. Use the test kit at temperatures between 15 and 30° C.
- Use the test kit between 10 and 90% humidity.
- Do not use the test kit beyond the expiration date (printed on the pouch and box).

NOTE: All expiration dates are printed in Day-Month-Year format.

WARNINGS PRECAUTIONS AND LIMITATIONS

- Not for medical or diagnostic use.
- Do not use after expiration date.
- Test device should remain sealed until ready for use.
- All samples must be considered potentially dangerous and therefore must be handled with the precautions of use relating to potentially infectious products.
- This kit contains products of animal origin. The certified knowledge of the origin and/or health status

of the animals does not fully guarantee the absence of transmissible pathogens. It is therefore recommended to treat these products as potentially infected and to handle them according to the usual safety precautions (do not ingest or inhale).

- Cross-contamination of samples shall be avoided by using a new sample collection container for each specimen obtained.
- Carefully read the entire procedure before performing any tests.
- Do not eat, drink or smoke in the area where samples and kits are handled.
- Observe established precautions against microbiological hazards throughout the procedure and follow the standard procedures for proper sample disposal.
- Wear protective clothing such as lab coats, disposable gloves and eye protection during sample analysis.
- Humidity and temperature can adversely affect the results.
- The materials used for testing must be manufactured in accordance with local, state and/or federal regulations.
- It is possible that technical or procedural errors, as well as other interfering substances in the urine can cause incorrect results.
- Adulterants, such as bleach and/or alum, may produce incorrect results in urine samples regardless of the analytical method used. If adulteration is suspected, the test should be repeated with another urine sample.
- A positive result indicates the presence of the drug or its metabolites but does not indicate the level of intoxication, route of administration or concentration in the urine.
- A negative result may not necessarily indicate drug-free urine. Negative results can be obtained when the drug is present but below the cut-off level of the test.
- The test is not able to distinguish between the drug and a medicine containing the same substance, false positives can be caused by some drugs.

SPECIMEN COLLECTION AND PREPARATION

1) Analysis of urine

The urine sample must be collected in the appropriate jar inside the clean and dry package. Urine can be collected at any time of the day. Urine samples with visible precipitates should be centrifuged, filtered or allowed to settle to obtain a clear sample for testing.


2) Storage of samples

Urine samples can be stored at 2-8 C up to 48 hours before analysis. For prolonged storage, samples may be frozen at a temperature below -20 C. Frozen samples must be thawed and mixed before testing.


TEST PROCEDURE

Allow the test, urine specimen, and/or controls to equilibrate to room temperature (15-30°C) prior to testing.

- 1) Remove the test device from the sealed bag and use it as soon as possible.
- 2) Place the test device on a flat surface and drive. Hold the dropper vertically and transfer 3 whole drops of urine (about 120µL) into the sample well (S) of the test device, then start the timer. Avoid trapping air bubbles inside the cockpit. See illustration below.
- 3) Wait for the red lines to appear. The result should be read after 5 minutes. Do not interpret the result after 10 minutes.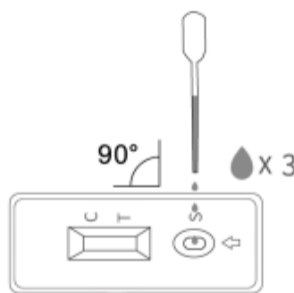

INTERPRETATION OF TEST RESULTS

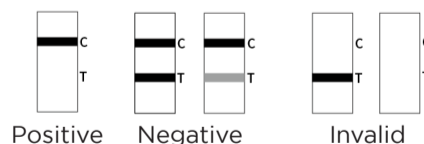POSITIVE: A colored line in the control line area (C) but no line in the test area (T) for a specific drug indicates a positive result. This indicates that the drug concentration in the sample exceeds the designated cut-off for that specific drug.

NEGATIVE: Two distinct colored lines appear. A colored line in the control line area (C) and a colored line in the test line region (T) for a specific drug indicates a negative result. This indicates that the drug concentration in the sample is below the threshold level designated for that specific drug. Note: The color gradient in the reactive zone (T) may vary, but must be considered negative whenever there is also a weak colored line.

INVALID: The control line (C) is not colored. The most likely reasons are insufficient sample volume or incorrect procedural techniques. Review the procedure and repeat the test using a new panel. If the problem persists, immediately stop using the test and contact your local distributor.

QUALITY CONTROL

Internal procedural controls are included in the test. A colored line appearing in the control region (C) is the internal procedural control. This procedural control line indicates that sufficient flow has occurred, and the functional integrity of the test device has been maintained. Control standards are not supplied with this kit; however, it is recommended that positive and negative controls be tested as a good laboratory practice to confirm the test procedure and to verify proper test performance.

Produced and distributed by:
Brands2China S.r.l.
brand owner:
Protezionipiù
Via Roma, 33
36014 - Santorso - Italia
www.protezionipiu.com